

Ophthalmic, sterile, moisturising and lubricating solution with natural lilac and chamomile extracts.

•  
Read the leaflet carefully.



## INDICATIONS

- Redness, dryness and fatigue of the eyes, prolonged work in front of a screen, tear deficiency, exposure to the sun, smoky or dry environments, or salt water.
- Prolonged wearing of contact lenses.
- Sensation of foreign bodies due to exposure to wind or dust.

## COMPOSITION

Sodium hyaluronate, glycerin, Malva sylvestris extract, Chamomilla recutita extract, sodium phosphate dibasic, sodium phosphate monobasic, sodium chloride, sodium chlorite, potassium hydroxide, purified water.

## DIRECTIONS FOR USE

2 to 3 drops, several times per day in each eye according to your doctor's instructions.

Put the bottle cap back on firmly after use.

## LEGAL NOTICES

This medical device is a regulated healthcare product that bears the CE marking under this regulation.

15 mL bottle

EAN code: 3770008240162

•  
Class IIb medical device

CE marking number: 0426

Manufacturer: Italdevice srl