

### Ophthalmic solution

KERADROP® is indicated for symptomatic treatment (itching, sensation of foreign objects, redness) during inflammation of the eye surface, such as eye discomfort, blepharitis and conjunctivitis, in both adults and children.

Read the leaflet carefully.



## PROPERTIES

- KERADROP® is a medical device composed of glycyrrhizin with phlogolytic activity (capable of inhibiting the progression of phlogosis).
- KERADROP® also contains a specific heat-sensitive copolymer which, by coming into contact with the surface of the eye and forming a gel, is able to provide long-term hydration and a lubricating effect.
- KERADROP® contains a new preservative system, which keeps the solution sterile and transforms into oxygen, water and sodium chloride as soon as it comes into contact with the surface of the eye. These substances, present naturally in tears, do not irritate the ocular mucosa.
- KERADROP® is therefore preservative-free in the eye.

## COMPOSITION

dipotassium glycyrrhizinate, sodium hyaluronate, PEG/PPG copolymer, EDTA disodium salt, sodium chlorite, potassium hydroxide, sodium phosphate monobasic, sodium phosphate dibasic, sodium chloride, purified water.

## DIRECTIONS FOR USE

Apply one or two drops into the affected eye(s), two or more times a day or according to medical advice.

## LEGAL NOTICES

This medical device is a regulated healthcare product that bears the CE marking under this regulation.

5 mL bottle

EAN code: 3401060037048

•

Class IIa medical device

CE marking number: 1370

Manufacturer: Italddevice srl